



SureMotion Wheelchair

User Manual

[Revision 1.0]

READ THIS MANUAL CAREFULLY BEFORE USE – FAILURE TO DO SO MAY RESULT IN INJURY, PROPERTY DAMAGE AND MAY VOID WARRANTY. • KEEP THIS MANUAL FOR FUTURE REFERENCE. • Products covered by this manual may vary in appearance, assembly, inclusions, specifications, description, and packaging.

THE BATTERY MAY HAVE SOME CHARGE ON ARRIVAL, IT IS IMPORTANT HOWEVER THAT THE BATTERY IS CHARGED PRIOR TO USE.

Safety

Safety messages are designed to alert you to possible dangers or hazards that could cause death, injury or equipment or property damage if not understood or followed. Safety messages have the following symbols:



You **WILL** be KILLED or SERIOUSLY INJURED if you do not follow instructions.

It is vital that you read and understand this user manual before using the product, including safety warnings, and any assembly and operating instructions. Keep the manual for future reference.

Safety precautions and recommendations detailed here must be fully understood and followed to reduce the risk of injury, fire, explosion, electrical hazard, and/or property damage.

Safety information presented here is generic in nature – some advice may not be applicable to every product. The term "equipment" refers to the product, be it electrical mains powered, battery powered or combustion engine powered.

- **Before Use** - If you are not familiar with the safe operation/handling of the equipment or are in any way unsure of any aspect of suitability or correct use for your application, you should complete training conducted by a person or organization qualified in safe use and operation of this equipment, including fuel/electrical handling and safety.
- Do NOT operate the equipment in flammable or explosive environments, such as in the presence of flammable liquids, gases, or dust. The equipment may create sparks or heat that may ignite flammable substances.
- Keep clear of moving parts.
- Equipment may be a potential source of electric shock or injury if misused.
- Do NOT operate the equipment if it is damaged, malfunctioning or is in an excessively worn state.
- Do NOT allow others to use the equipment unless they have read this manual and are adequately trained.
- Keep packaging away from children - risk of suffocation! Operators must use the equipment correctly. When using the equipment, consider conditions and pay due care to persons and property.

General Work Area Safety

- Work areas should be clean and well lit.
- Do not operate the equipment if bystanders, animals etc are within operating range of the equipment or the general work area.
- If devices are provided for connecting dust extraction / collection facilities, ensure these are connected and used properly. Dust collection can reduce dust-related hazards.



You **CAN** be KILLED or SERIOUSLY INJURED if you do not follow instructions.

General Personal Safety

- Wear appropriate protective equipment when operating, servicing, or when in the operating area of the equipment to help protect from eye and ear injury, poisoning, burns, cutting and crush injuries. Protective equipment such as safety goggles, respirators, non-slip safety footwear, hard hat, hearing protection etc should be used for appropriate equipment / conditions. Other people nearby should also wear appropriate personal protective equipment. Do not wear loose clothing or jewellery, which can be caught in moving parts. Keep hair and clothing away from the equipment.
- Stay alert and use common sense when operating the equipment. Do not over-reach. Always maintain secure footing and balance.
- Do not use the equipment if tired or under the influence of drugs, alcohol, or medication.
- This equipment is not intended for use by persons with reduced physical, sensory, or mental capabilities.

General Fuel Safety

- Petrol/fuel/gasoline is extremely flammable – keep clear of naked flames or other ignition sources.
- Do not spill fuel. If you spill fuel, wipe it off the equipment immediately – if fuel gets on your clothing, change clothing.
- Do NOT smoke near fuel or when refuelling.
- Always shut off the engine before refuelling.
- Do NOT refuel a hot engine.
- Open the fuel cap carefully to allow any pressure build-up in the tank to release slowly.
- Always refuel in well-ventilated areas.
- Always check for fuel leakage. If fuel leakage is found, do not start, or run the engine until all leaks are fixed.

General Carbon-Monoxide Safety

- Using a combustion engine indoors **CAN KILL IN MINUTES**. Engine exhaust contains carbon-monoxide – a poison you cannot smell or see.
- Use combustion engines OUTSIDE only, and far away from windows, doors, and vents.



You **CAN** be INJURED if you do not follow instructions or equipment damage may occur.

General Equipment Use and Care

- Keep packaging away from children and pets, potential risk of suffocation or choking.
- The equipment is designed for domestic use only.
- Handle the equipment safely and carefully.
- Before use, inspect the equipment for misalignment or binding of moving parts, loose components, damage, or any other condition that may affect its operation. If damaged, have the equipment repaired by an authorised service centre or technician before use.
- Prevent unintentional starting of the equipment - ensure equipment and power switches are in the OFF position before connecting or moving equipment. Do not carry equipment with hands or fingers touching any controls. Remove any tools or other items that are not a part of the equipment from it before starting or switching on.
- Do not force the equipment. Use the correct equipment for your application. Equipment will perform better and be safer when used within its design and usage parameters.
- Use the equipment and accessories etc. in accordance with these instructions, considering working conditions and the work to be performed. Using the equipment for operations different from those intended could result in hazardous situations.
- Always keep equipment components (engines, hoses, handles, controls, frames, housings, guards etc) and accessories (cutting tools, nozzles, bits etc) properly maintained. Keep the equipment clean and, where applicable, properly lubricated.
- Store the equipment out of reach of children or untrained persons. To avoid burns or fire hazards, let the equipment cool completely before transporting or storing. Never place or store the equipment near flammable materials, combustible gases, or liquids etc.
- The equipment is not weather-proof, and should not be stored in direct sunlight, at high ambient temperatures or locations that are damp or humid.
- Do not clean equipment with solvents, flammable liquids, or harsh abrasives.
- For specific equipment safety use and care, see Equipment Safety.

General Electrical Safety

- Inspect electrical equipment, extension cords, power bars, and electrical fittings for damage or wear before each use. Repair or replace damaged equipment immediately.
- Ensure all power sources conform to equipment voltage requirements and are disconnected before connecting or disconnecting equipment.
- When wiring electrically powered equipment, follow all electrical and safety codes.
- Always use a Residual Current Device (RCD) (GFCI) / Earth Leakage Circuit Breaker / Safety Switch when operating this item (electrical items only).
- High voltage / high current power lines may be present. Use extreme caution to avoid contact or interference with power lines. Electrical shock can be fatal.

General Electrical Safety

- Electrically grounded equipment must have an approved cord and plug and be connected to a grounded electrical outlet.
- Do NOT bypass the ON/OFF switch and operate equipment by connecting and disconnecting the electrical cord.
- Do NOT use equipment that has exposed wiring, damaged switches, covers or guards.
- Do NOT use electrical equipment in wet conditions or in damp locations.
- Do NOT use electrical cords to lift, move or carry equipment.
- Do NOT coil or knot electrical cords, and ensure electrical cords are not trip hazards.

General Service Information

- The equipment must be serviced or repaired at authorised service centres by qualified personnel only.
- Replacement parts must be original equipment manufacturer (OEM) to ensure equipment safety is maintained.
- Do NOT attempt any maintenance or repair work not described in this manual.
- After use, the equipment and components may still be hot – allow the equipment to cool and disconnect spark plugs and/or electrical power sources and/or batteries from it before adjusting, changing accessories, or performing repair or maintenance.
- Do NOT adjust while the equipment is running.
- Perform service-related activities in suitable conditions, such as a workshop.
- Replace worn, damaged or missing warning/safety labels immediately.

Child Choking Hazard!

• WARNING: CHOKING HAZARD - CHILDREN UNDER 8 YRS. CAN CHOKE OR SUFFOCATE ON SMALL PARTS. ADULT SUPERVISION REQUIRED.

- Carefully inspect anything a baby or child could gain access to, including this product & packaging.
- Dangers include anything of a size that could become stuck or lodged in a baby or child's airway.
- Dangers include things like raw, hard fruit and vegetable pieces, large pieces of meat, bones or sausage skins, popcorn, nuts, hard lollies and corn chips, small magnets and batteries, coins, beads, marbles and small uninflated balloons, broken toys and smaller toys, buttons, keys, and coins. But there are also many more.
- Pay particular attention to accessories, nuts, bolts, screws, washers, caps, covers, loose parts, parts that could be removed or break off, pieces of packaging, staples, tape etc.
- In the case of any doubt, secure the item of concern in an appropriate manner, or completely remove the possibility of access by a baby or child.

Important Wheelchair Safety

- When attempting to stand you must ensure the footrests have been flipped up prior to attempting to exit the chair.
- Assistance may be required to exit the chair. A qualified carer is recommended when traveling up or down an incline.
- Do not stand directly on the wheels, do not place any weight directly on the wheels.
- When sitting on the seat ensure the brakes are engaged and the user is balanced towards the back of the seat. Do not lean forward.
- The wheelchair is not designed for transiting a patient through long distances or for use on rough, uneven surfaces.
- Never attempt to travel up or down an incline without assistance if the user has reduced strength, balance, or cognitive awareness.
- Do not lean forward when travelling downhill or backward when travelling uphill.
- Sudden shifts in body weight may cause the wheelchair to tip forward or backward.
- Always maintain an upright seated position when moving on slopes or uneven surfaces.
- Do not allow the wheelchair to roll freely downhill.
- Use extra caution on wet, loose, or uneven surfaces.
- Do not park on an incline.

Other Wheelchair Safety Reminders

- Please read and follow all instructions, warnings, and notes in this manual before attempting to operate your power wheelchair for the first time. If there is any information in this manual which you do not understand, or if you require additional assistance for assembly or operation, please contact your authorised local provider. The safe use of your product depends on whether you follow the instructions, cautions, and warnings in this manual. We are not liable for any damage and/or injuries resulting from unsafe operation or failure to follow the instructions, cautions, and warnings in this manual. The symbols below in this manual are used to identify warnings and important information. All of them are very important for your safety. It is strongly recommended that you read and understand them completely.
- Please read the user manual carefully and follow riding requirements during the power chair ride. Do not use it until riders have read the user manual carefully and learned the performance well. Do not lend it to those who cannot operate the power chair.
- You need lots of practice to grasp the skill before you ride the power chair safely. If you are not skilled or fail to follow the user guide of this product, it may cause injury to the rider or others, or property loss. Our company is only responsible for product defects or damage, and we are NOT responsible for physical injury or property loss related to using the product.

Attentions Before Operation

- Get familiar with local traffic rules.
- The power chair is classified as a non-motor vehicle, designed especially for the elderly and people with mobility issues. No driving licence is needed in accordance with traffic law, but the driver of the power chair should observe the same traffic rules that apply to pedestrians.
- Please drive the power chair in pedestrian lanes or bike lanes.
- Get familiar with all the traffic signals and marks used for pedestrians.
- Do not change direction or cross the road suddenly when driving.

Safety Symbols

The product may have safety warning labels attached to it, explained below. Understand the symbols on your product and their meanings. If any stickers become unreadable, unattached etc., replace them.

 Flammable Material Hazard Flammable liquids, gases, or substances etc may present. Avoid ignition sources and open flames. Danger of fire.	 Read User Manual Read and fully understand product safety warnings, operation, procedures etc before using the product.	 Use Hand Protection Wear appropriate hand protection and take due care as the product or use of the product may present hand hazards.	 Carbon-Monoxide Hazard Do not use the product in confined areas or without adequate ventilation. Carbon-monoxide poisoning can be fatal.
 Electrocution / Electrical Shock Hazard High voltage or high current electricity may be present or required by the product. Take due care when handling electrical products, cables, plugs and leads. Electrical shock can be fatal.	 Toxic Fumes / Dust Hazard Using the product or by-products from use may produce fumes, smoke or particles that could be harmful if inhaled. Wear appropriate breathing protection and have adequate ventilation.	 Explosive Material Hazard Combustible liquids, gases, or substances etc may be present. Avoid ignition sources and open flames. Danger of explosion.	 Cutting / Amputation Hazard The product may have blades, edges or mechanical devices that can cause severe cut injury to fingers, limbs etc. Take due care when handling and using the product.
 Crush Hazard The product may have blades, edges or mechanical devices that can cause severe crush injury to fingers, limbs etc. Take due care when handling and using the product.	 Single Operator Only The product must be operated by a single person only. More than one person operating the product may introduce additional hazards.	 Use Face Protection Wear appropriate full-face protection and take due care as the product or use of the product may present face and eye hazards.	 Use Foot Protection Wear appropriate foot protection and take due care as the product or use of the product may present foot hazards.
 Use Eye / Ear / Head Protection Wear appropriate eye and / or ear and / or head protection and take due care as the product or use of the product may present eye, hearing, and head hazards.	 Running Hazard Do not run on or near the product as doing so may present a fall hazard.	 Diving Hazard Do not dive into the product as doing so may present a neck / head injury hazard.	 Adult Supervision Required Always supervise children and other users of a product to prevent drowning or injury.
 Skin Penetration / Puncture Hazard The product may produce pressure, emit liquids or objects that can cause severe injury to fingers, limbs, blood etc. Take due care when handling and using the product.	 Hot Surface Hazard Be aware that the product may produce high temperatures and hot surfaces that can cause burn injuries.	 Flying Debris Hazard Be aware that the product or use of the product may present hazards produced by flying debris. Wear appropriate clothing and protective devices.	 Moving Parts Hazard Be aware that the product contains or uses mechanical devices that move or rotate. Always wait for moving parts to stop fully before handling the product, adjusting, maintenance etc.

 Carbon-Monoxide Hazard Do not use the product in confined areas or without adequate ventilation. Carbon-monoxide poisoning can be fatal.	 Pull Hazard Be aware that the product contains or uses mechanical devices that can pull in objects and can cause severe injury to fingers, limbs etc. Take due care when handling and using the product.	 Slope / Fall Injury Hazard Be aware that using the product on sloping surfaces or in slippery conditions may present additional dangers from falls and contact with blades, moving parts, hot surfaces etc.	 "Slam Dunk" Warning Do NOT attempt "slam dunk" manoeuvres as this may result in severe injury due to falling, product breakage or collapse etc.
 Electrocution / Electrical Shock Hazard - Outdoor High voltage or high current electricity may be present or required by the product. Do NOT use in rain, damp, or wet conditions. Electrical shock can be fatal.	 Electrocution / Electrical Shock Hazard - Disconnect High voltage or high current electricity may be present or required by the product. Always disconnect the product from the electrical supply before handling the product, adjusting, maintenance etc.	 Power Line Electrocution Hazard High voltage / high current power lines may be present. Use extreme caution to avoid contact or interference with power lines. Electrical shock can be fatal.	 "Kick-Back" Hazard High level of "kick-back" hazard that can cause the machine to suddenly rotate towards operator. Kick-back injury can be fatal.
 Winch Operator Position Hazard Do NOT stand between winch and load. Do NOT use winch to move people.	 Winch Lift Hazard Do NOT LIFT load vertically. Use machine to PULL only.	 Cable Hazard Ensure that load bearing cable is not kinked or knotted.	 Winch Cable Hazard Ensure that there is a minimum number of cable coils on winching mechanism.
 Winch Hook Hazard Carry hook to load – do NOT throw or run.	 Flash / Blinding Hazard Wear appropriate eye protection for welding. Direct exposure to weld arcs may cause permanent eye injury.	 Laser Hazard Laser may be in use – do NOT look directly at laser or allow others to.	 Choking Hazard Children under 8 yrs. Can choke or suffocate on small parts. Adult supervision required.

Table of Contents

Safety	2
Safety Symbols	5
Parts Identification	8
Operation	9
Check the Battery Level	9
How to Charge the Wheelchair	10
Using the Electric Stair Climbing Wheelchair	11
Power On	11
Adjusting the Hand Pull Bar	11
Folding and Unfolding the Track Assembly	12
Unfolding the Front Handrail	12
Rear Handrail	13
Footrest	13
Emergency Rod	14
Brake	14
Power On	15
Controller Operation	15
Use of Additional Assistance	15
Lithium Battery Care and Maintenance	16
Top 3 Battery Care Requirements	16
Maintenance	18
Specifications	19

Parts Identification



No.	Name	No.	Name
1	Control Panel	8	Front Wheel
2	Lift Bar	9	Footrest
3	Rear Handle	10	Front Handle
4	Track Assembly	11	Lithium Battery
5	Controller	12	Body Safety Belt
6	Motor	13	Armrest
7	Rear Wheel		

Operation

Check the Battery Level

1. Turn the battery key clockwise fully until it stops.



2. Press the red-and-white button. If the battery has power, the green indicator lights will turn on.

- **One light on** = Battery level is 30%
- **Two lights on** = Battery level is 50%
- **Three lights on** = Battery level is 100%
- **Four lights on** = Battery level is below 10%




How to Charge the Wheelchair



1. Unscrew the battery connection cable anticlockwise.



2. Insert the charger cable and charge the battery.

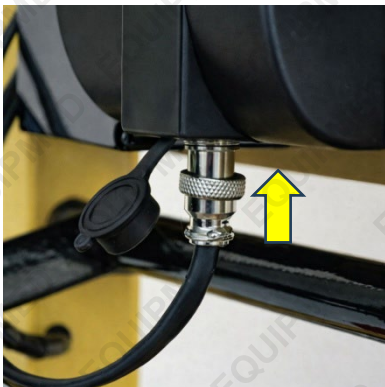

3. Once the battery is fully charged, remove the charger cable. Reinsert the connection cable and tighten it clockwise by hand until secure.



4. To remove the battery, press down on the battery key and turn it anticlockwise fully, then lift the battery out.

Using the Electric Stair Climbing Wheelchair

Power On

Turn the key clockwise to the ON position. When the green indicator light turns on, the power is on.



Adjusting the Hand Pull Bar

To extend the hand pull bar, press the red horizontal bar and lift the hand pull bar upwards.

There are four adjustable height positions:

- First height: 103 cm
- Second height: 123 cm
- Third height: 138 cm
- Fourth height: 156 cm




Folding and Unfolding the Track Assembly

1. Push the lower rod, as indicated by the red arrow, forwards in the direction shown to unlock the track locking mechanism.
2. Adjust the track assembly to the required position.
3. There are three adjustment positions in total.



Unfolding the Front Handrail

To unfold the front handrail, press it downwards to the fixed position. Once secured, the front handrail can be used.

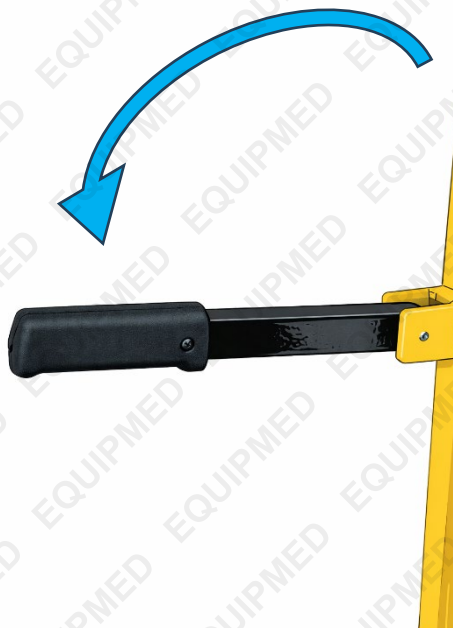


Rear Handrail

Unfold the rear handrail:

Push the rear handrail upwards until it locks into position.

To fold the rear handrail:

Press and hold the red button, then push the rear handrail downwards to fold it.

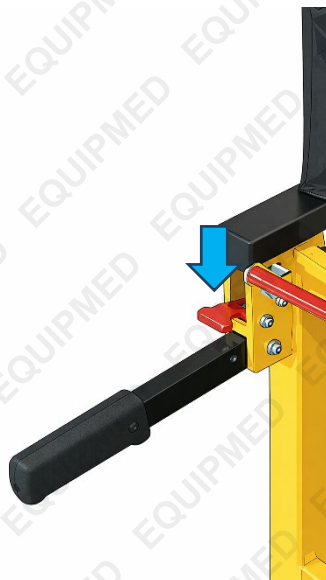


Footrest

Ensure the footrest is fully lowered and securely positioned before use.



Emergency Rod

Press the red buttons on both sides simultaneously, then pull the rod outwards to activate it.

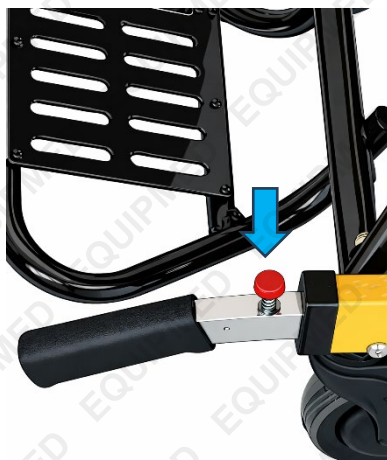

Brake

Pull down or lift up the black brake lever to apply or release the brake on the stair-climbing wheelchair.



Power On

Turn the key clockwise fully until it stops to switch the unit on. The front indicator lights on the battery will illuminate.



Controller Operation



- Press the red button to turn the power on.
- Press the yellow button to turn the lights on.
- The “+” button increases the operating speed.
- The “-” button decreases the operating speed.
- UP indicates ascending stairs.
- DOWN indicates descending stairs.

Use of Additional Assistance

It is recommended that two operators are present when using the stair-climbing wheelchair to evacuate individuals weighing more than **169 kg**.

Lithium Battery Care and Maintenance

Top 3 Battery Care Requirements

1. **Charge battery before first use.** The battery may arrive with some charge, regardless of this the battery must be charged on the initial charge and for each subsequent charge. ✓
2. **Charge battery immediately when it is depleted.** Do not store a depleted battery. This will permanently impact the life of the battery. If the battery is going to be stored for a period of time, fully charge the battery before storing. ✓
3. **Disconnect battery when not in use.** To prolong the life of a battery, disconnect it from the unit when not in use. Disconnect chargers from the battery once charged. ✓

To ensure the battery maintains optimum performance, follow the guidelines outlined below.

- For best results, charge batteries in temperatures between 10°C and 30°C
- When battery is charged, disconnect the charger from the power supply and remove the battery from the charger.
- If a battery is stored or otherwise unused for an extended period, and the battery has no charge remaining when you check it, consider it to be damaged. Do not attempt to recharge it or to use it. Replace it with a new battery.
- Use only with the battery charger specified by the manufacturer.
- Ensure that the battery charger electrical cord is not subjected to damage or stress. Do not operate the battery charger if it has a damaged electrical cord or plug.
- Do not use the charger in wet areas or expose it to rain or water.
- The typical estimated life of a Lithium-Ion battery is about two to three years or 300 to 500 charge cycles, whichever occurs first. One charge cycle is a period of use from fully charged, to fully discharged, and fully recharged again.
- Explosion hazard – protect the battery against heat, for example, direct sunlight and fire. Do not store batteries in vehicles or locations subject to heat.
- Batteries can explode in the presence of an ignition source, such as a pilot light. To reduce the risk of serious personal injury, never use any cordless product in the presence of open flame. Exploding batteries can propel debris and chemicals.
- Do not open the battery – danger of short-circuiting and/or explosion.
- Do not attempt to destroy or disassemble the battery pack or remove any of its components.
- Do not touch the battery terminals with metal objects and/or body parts as short-circuit and/or personal injury may result.
- Explosion hazard – do not open and/or short-circuit the battery.
- Poison hazard – battery leakage (liquid ejection). Under extreme conditions, liquid may be ejected from the battery – avoid contact. If contact accidentally occurs, immediately seek medical attention, and flush with water. If liquid contacts eyes, immediately seek medical attention, flush with clean water for at least 10 minutes. Liquid ejected from the battery may cause irritation or burns.
- Dispose of non-serviceable batteries in an environmentally responsible manner and according to local regulations.

Failure to adhere to the requirements above may result in the battery becoming inoperative or battery life being reduced. In extreme cases or abuse, battery overheating, or fire may result.

DANGER - BATTERY CHARGING, USE AND STORAGE

Like mobile phones, laptops, cordless vacuum cleaners etc, this product may contain or be related to a lithium ion battery pack (lithium ion batteries and lithium ion battery related items only). Lithium ion batteries allow fast charging and high capacities, however care must be taken to reduce chances of a fault, and increase safety in case a fault occurs. Catastrophic incidents involving lithium ion batteries have a tendency to result in large amounts of smoke and fire. Be sure to follow the points below to reduce the likelihood of danger whenever you use, charge or store any lithium ion battery.

- Never charge a battery that has been damaged or subjected to a forceful impact.
- Never charge a battery that has been exposed to moisture or water.
- Never charge a battery with a charger that wasn't supplied with the battery.
- Never use a battery in a device that it wasn't specifically designed for.
- Never use a battery with a modified or damaged device.
- Never modify the battery, product, charger, circuitry or wiring.
- Never charge a battery in an exit path that may be used by people to leave a room or building in case of an emergency, for example, the hallway or foyer of a residence.
- Never charge a battery in an area or building that doesn't have the recommended number of working smoke detectors that have been tested in accordance with local guidelines.
- Never charge a battery when the battery or charger is displaying a problem or charging incorrectly.
- Always charge batteries on a non-combustible surface, for example, on a concrete floor.
- Always charge batteries at least 3m away from anything combustible, such as floors, wall surfaces, furniture, storage shelves etc.
- Always charge batteries in an open area (outside if possible) ensuring the battery and charger are away from moisture and water.
- Always disconnect batteries from the charger after charging.
- Always remove or disconnect batteries from the device/product when not in use.
- Always ensure batteries are charged under adult supervision. Do not charge unattended. Do not charge batteries while no one is home or while people are sleeping.
- Keep batteries away from heat and moisture.
- Damaged batteries should be disposed of through your local e-waste collection service.
- Avoid mixing and matching chargers. Unplug products when fully charged, and charge batteries in a cool, dry place away from combustible materials like beds, lounges, or carpet.
- During charging and use, check your lithium-ion batteries for overheating, signs of swelling, leaking, or venting gas. Immediately stop using your product if these signs are present.
- Allow batteries to cool after use and before charging.

Maintenance

- Clean the stair-climbing wheelchair and its components using a clean cloth or soft brush with warm, soapy water.
- Rinse with clean warm water and allow to air dry.
- Do not use cleaning products containing bleach or phenol.
- A stiff-bristled brush may be used if necessary.

WARNINGS

- **NEVER LEAVE AN EVACUEE UNATTENDED IN THE STAIR-CLIMBING WHEELCHAIR. ALWAYS USE THE SAFETY RESTRAINTS.**
- **AVOID CHANGING POSITION RAPIDLY, AS THIS MAY CAUSE INJURY TO THE EVACUEE. SUPPORT THE STAIR-CLIMBING WHEELCHAIR WHILE ADJUSTING THE HANDLES.**
- **IMPROPER MAINTENANCE MAY CAUSE INJURY. MAINTAIN AND USE THE STAIR-CLIMBING WHEELCHAIR ONLY FOR THE PURPOSES DESCRIBED IN THIS MANUAL.**

Specifications

Power (Watts)	120W
Power Type	Battery
Battery Type	Lithium Ion
Battery Capacity	13Ah
Battery Voltage	24V
Climbing Speed	Up to 1.3 seconds per step (46 steps per minute)
Descending Speed	Up to 1.1 seconds per step (55 steps per minute)
Maximum Incline	≤50°
Max User Load	169 kg
Runtime	Up to 500 steps
Frame & Chassis Material	Aluminium alloy
Corrosion Resistant	Yes
Finish	Powder coated
Seat / Backrest Cushion Material	Cloth
Front Wheels	5"
Rear Wheels	10"



Some experts believe that the incorrect or prolonged use of almost any product may cause serious injury or death. To help reduce your risk of serious injury or death, refer to the information below. For more information, see www.datastreamserver.com/safety

- Consult all documentation, packaging, and product labelling before use. Note that some products feature documentation available online. It is recommended to print and retain the documentation.
- Before each use, check the product for loose/broken/damaged/missing parts, wear, or leaks (if applicable). Never use a product with loose/broken/damaged/missing parts, wear, or leaks.
- Products must be inspected and serviced (if applicable) by a qualified technician every 6 months. This is based on average residential use by persons of average size and strength, and on a property of average metropolitan size. Use beyond these recommendations may require more frequent inspections/servicing.
- Ensure that all users of the product have completed a suitable industry recognised training course before being allowed access to the product.
- The product has been supplied by a general merchandise retailer that may not be familiar with your specific application or description of application. Be sure to attain third-party approval from a qualified specialist for your application before use, regardless of any assurances from the retailer or its representatives.
- This product is not intended for use where fail-safe operation is required. As with any product (for example, automobile, computer, toaster), there is the possibility of technical issues that may require the repair or replacement of parts, or the product itself. If the possibility of such failure and the associated time it may take to rectify could in any way inconvenience the user, business, or employee, or financially affect the user, business, or employee, then the product is not suitable for your requirements. This product is not intended for use where incorrect operation or a failure of any kind, including but not limited to, a condition requiring product return, replacement, parts replacement, or service by a technician may cause financial loss, loss of employee time or an inconvenience requiring compensation.
- If this product has been purchased in error when considering the information presented here, contact the retailer directly for details of their returns policy, if required.

