


# Transfer chair

---

# User manual


**Model: MN-HY801**

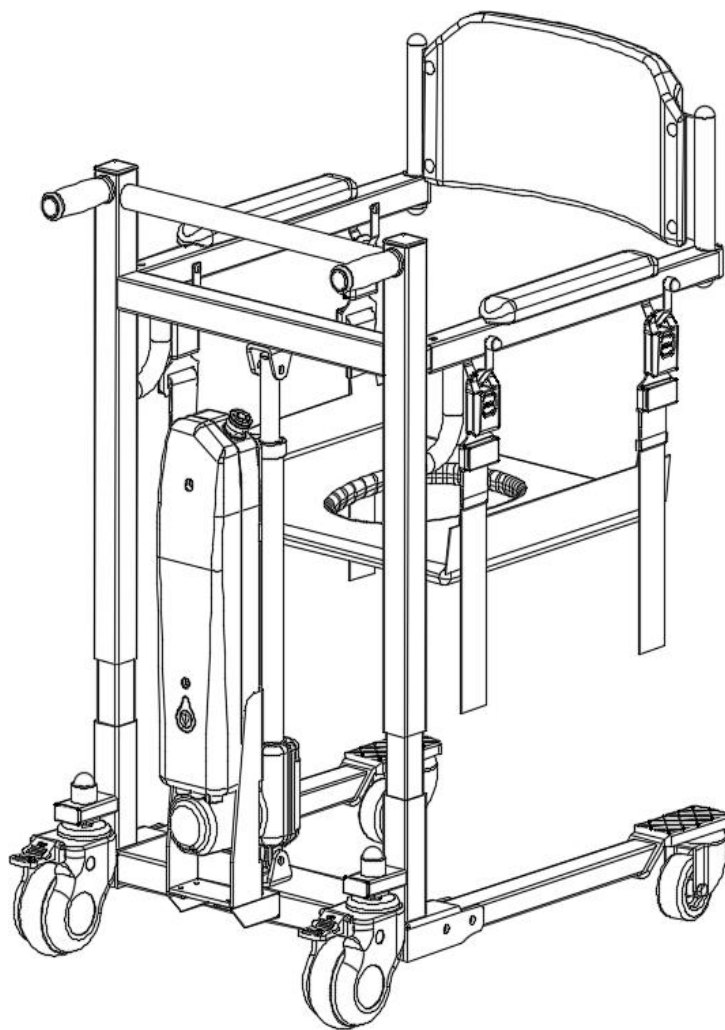

# Contents

|                                     |   |
|-------------------------------------|---|
| I. Parts introduction.....          | 1 |
| II. Technical specification.....    | 1 |
| III. Installation instructions..... | 2 |
| IV. User guide.....                 | 4 |
| V. Troubleshooting.....             | 6 |
| VI. Warranty.....                   | 6 |


# I. Parts introduction

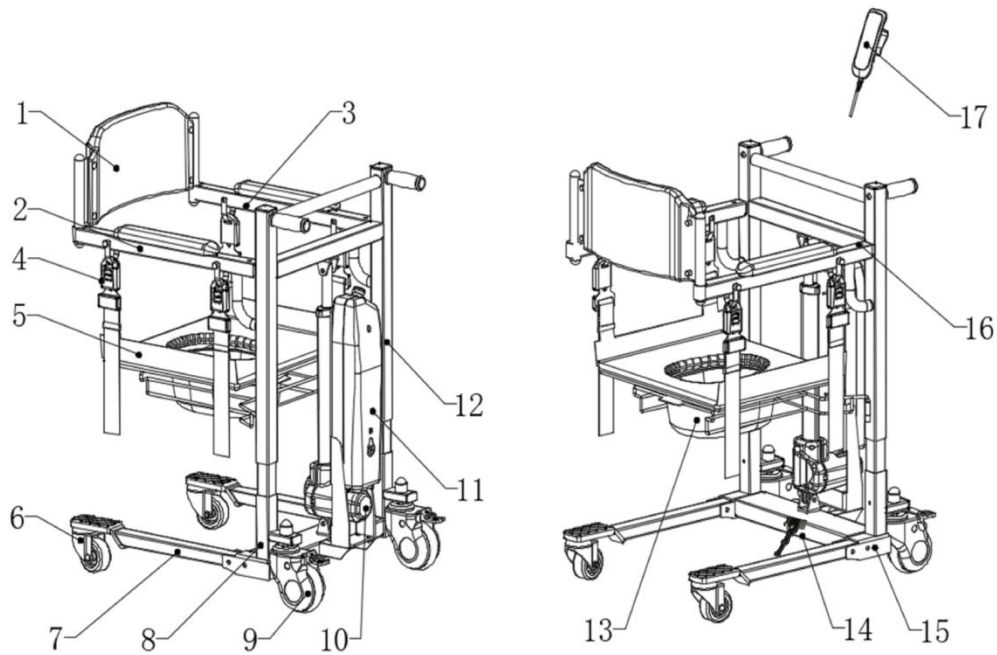

- 1、 Backrest; 2、 Right armrest; 3、 Left armrest; 4、 Ring; 5、 Cushion;  
 6、 3" front wheel; 7、 Support leg; 8、 Base; 9、 5"Rear wheel(with brake);  
 10、 Motor; 11、 Controller box/ battery; 12、 Frame; 13、 Bedpan;  
 14、 Arm-brace; 15、 Long bolt; 16、 Short bolt; 17、 Operation panel


# II. Technical specification

|                               |                      |
|-------------------------------|----------------------|
| Overall size                  | 770*550*870mm        |
| Net weight                    | 35 Kg                |
| Lifting range                 | 0~250mm              |
| Lifting speed                 | 7.6mm/s              |
| Motor                         | DC24V actuator motor |
| Rated power                   | 72W                  |
| Charger                       | AC220V/50Hz          |
| Loading capacity              | 150Kg                |
| Noise                         | 65 (dB)              |
| Lifting times per full charge | 60 Times             |

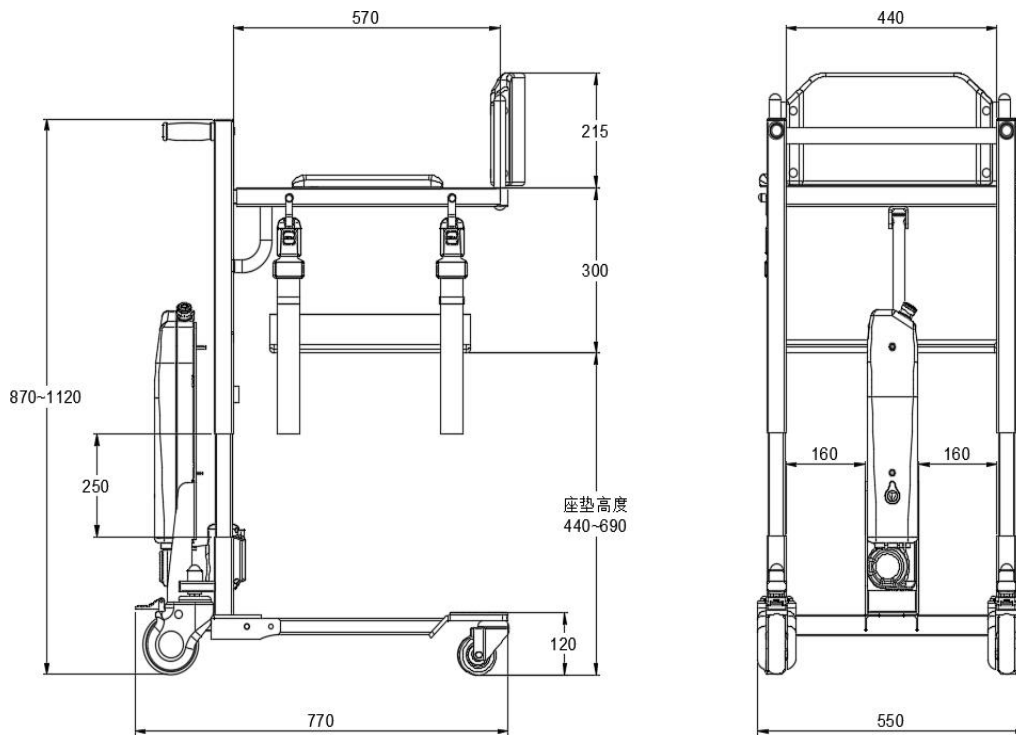


### III. Installation instructions

#### 1. Parts

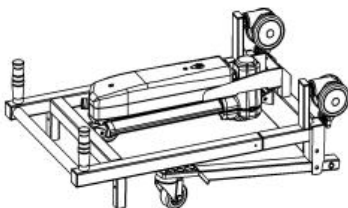

Frame x 1 set

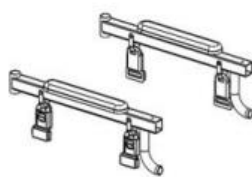

Armrest 1 pair



Bedpan 1 PC




Spare parts 1 set




Backrest x 1



(Optional backrest  
x 1)

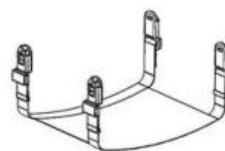


Soft cushion 1 PC

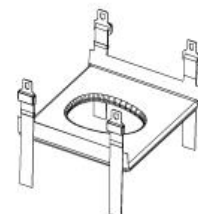


Hard cushion x 1

## 2. Installation steps


### Step 1: install arm-brace

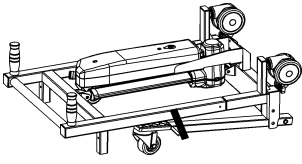

① Take out the body

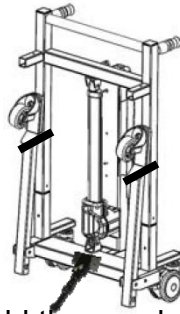


② unfold the arm-brace

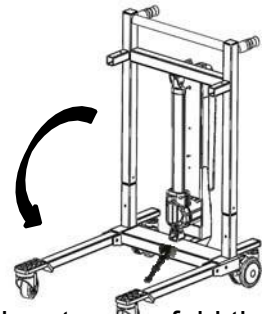


③ Untie the strap, unfold the support leg



④ Insert the long bolt into the hole

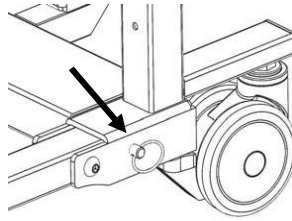


⑤ Install bolts on both sides of the same operation

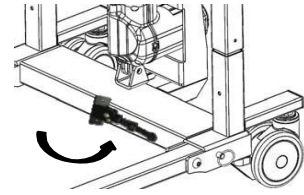


⑥ Fold the arm-brace


### Step 2: Install armrest

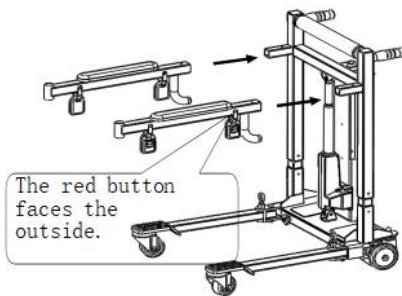


① Distinguish the left and right armrests.

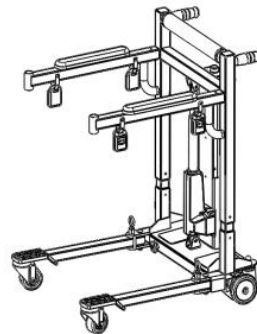

② Insert the two armrests into the main frame.

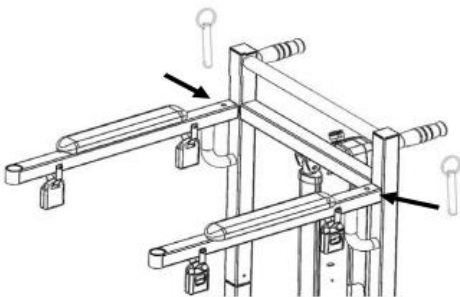

③ Insert the short bolt into the hole

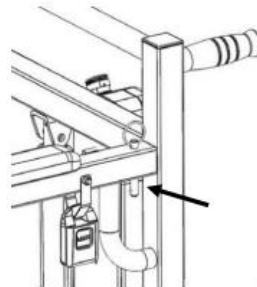


④ complete

# IV. User guide

## 1. How to use the controller panel

Press the up and down button to control the machine, release the button to stop the movement.

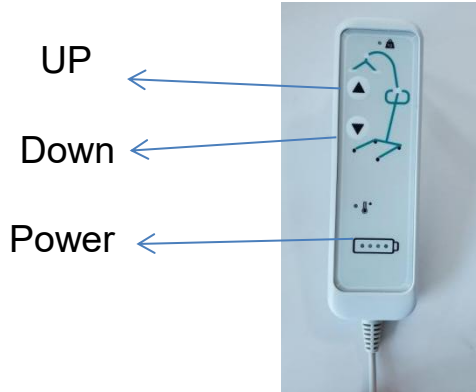


## 2. Emergency stop

The emergency stop button is used to interrupt and turn on the power. so that the motor will stop working immediately in the case of danger. Stop working immediately in case of danger.

(Close: Press the red button, stop working.①)


Open: Rotate the red button clockwise, normal working.②

**Suggestion: If the machine has not been used for more than 3 days, push this button to protect the battery.**

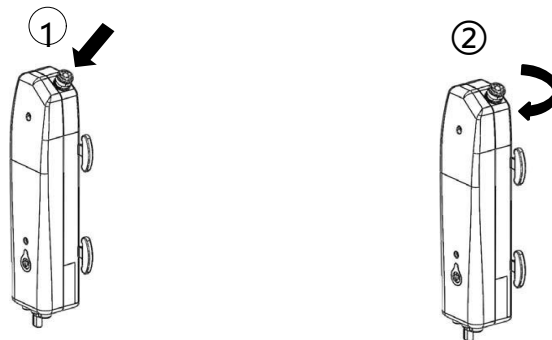

## 3. Control device connection


There were 4 sockets.

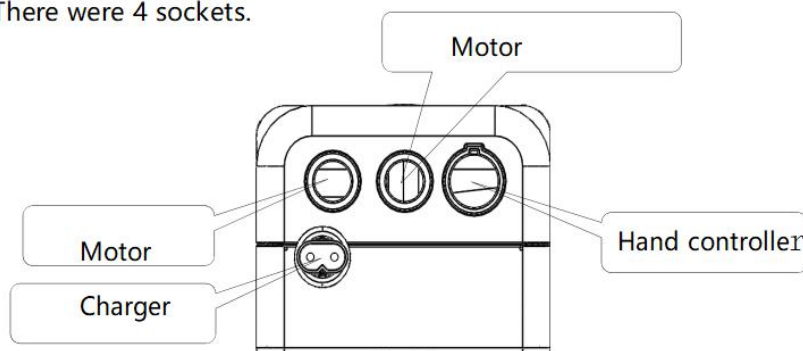

## 4. Front wheel brake



## 5. Operation of the transfer chair.

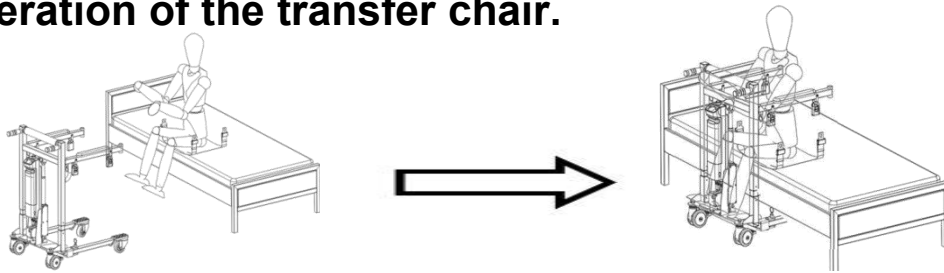


①. Sit on the cushion


②. Push into , fasten the 4 rings



Shower

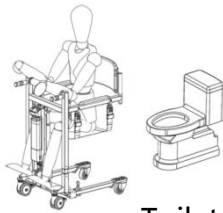


Toilet

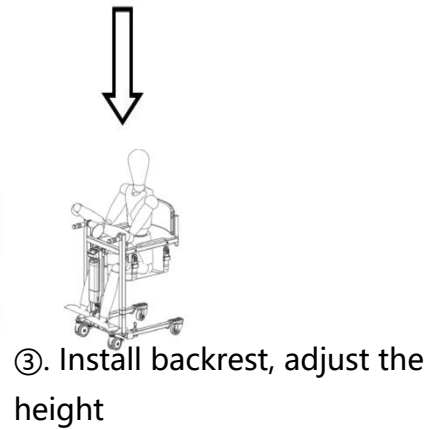

## 6. Storage of transfer chair


Folding steps:

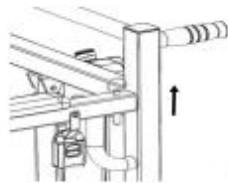

① Pull out the bolt from the armrest



② Remove the armrests

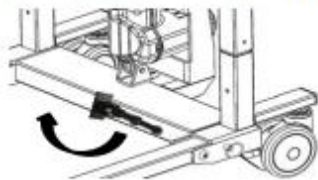

③ Unfold the arm-brace




④ Pull out the bolt, fold support leg



⑤ Complete

## 7. Battery charge.

When the battery power is only 1 grid, it needs to be charged immediately. If not, the motor will stop working.

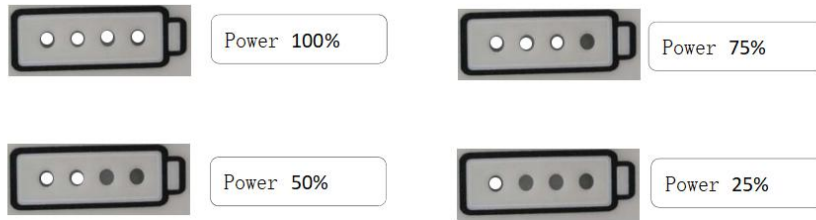

### Notes:

1. Please close the emergency button and turn off the machine when charging.
2. It is generally charged once every month when the machine is not used for a long time.
3. Do not charge in a humid environment.
4. If the shifter is not used every day, it is recommended to close the emergency stop button.

## V. Troubleshooting

|                        |   |
|------------------------|---|
| Couldn't lift or land  | <ol style="list-style-type: none"> <li>1. Check whether the emergency stop button is pressed</li> <li>2. Check whether the wire of is connected correctly</li> <li>3. Check battery power</li> <li>4. Check whether the battery contact panel has problems or damages</li> <li>5. If the electric shift machine is not working, please contact the after-sales service</li> </ol> |
| Charging failure       | <ol style="list-style-type: none"> <li>1. Check whether the charger is plugged into the socket</li> <li>2. Check the battery contact panel for problems or damage</li> <li>3. If the charger still does not charge, please contact after-sales service</li> </ol>   |
| It stops while lifting | <ol style="list-style-type: none"> <li>1. Check whether the emergency stop button is pressed</li> <li>2. Use an emergency descent device</li> <li>3. If the electric shift machine is not working, please contact the after-sales service.</li> </ol>   |

## VI. Warranty

| Warranty Card      |      |                    |          |
|--------------------|------|--------------------|----------|
| User Name          |      | Phone              |          |
| Address            |      |                    |          |
| Product Name       |      | Model              |          |
| Serial Number      |      |                    |          |
| Maintenance Record | Date | Causes and Handing | Repairer |
|                    |      |                    |          |
|                    |      |                    |          |